

# Alumina powder instruction manual

**【Product description】**

It consists of alumina and is a fine white powder.

**【Character】**

This product is white, odorless, tasteless powder.

**【Specification】**

27um, 50um, 90um

**【Expected use】**

It is used outside the mouth to grind and polish the prosthetics so that the surface is smooth and even. This device is for professional personnel only. Wear rubber medical gloves and protective clothing when operating the device.






**【Instructions】**

When it is used, it is put into the sandblasting tank of the sandblasting machine. Through the air compression of the sandblasting machine, the air flow drives the sandblasting powder (alumina particles) to directly act on the surface of the dental crown, inlaid and orthodontic fasteners, etc., for the external cleaning and micro-roughening of the dental prostheses before bonding.

**【General statement】**

- 1. Taboo
  - 1.1 People who are allergic to any component of the product should not be allowed;
  - 1.2 Please follow the doctor's advice and use it under the doctor's guidance;
  - 1.3 Do not use this product if its character has changed;
  - 1.4 Do not swallow, keep in a place inaccessible to children;
  - 1.5 If swallowed in large quantities, seek medical attention immediately.
- 2. Adverse reactions are unclear.
- 3, production date see label.
- 4, shelf life of 24 months.
- 5. Store in a cool and dry environment and avoid light.

**【Symbol description】**

	Please read the instructions		Avoid the sun
	Batch code		Manufacturers
	Registered trademark		



**Guilin Yikeshi Medical Device Co., LTD. (XPEDENT LTD)**

D-8, Guilin National High-tech Zone Information Industrial Park,  
Chaoyang Road Qixing District, Guilin, Guangxi, China 541004  
Tel: +86 773 5805522  
Fax: +86 773 5805522  
mail: sales@xpedency.cn  
Website: www.xpedent.com.cn

